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# Information leaflet for URITAM tampons

EN

**URITAM** tampons are intended for women who experience discomfort associated with:

- urinary incontinence, especially stress incontinence,
- lowering of the reproductive organs.

Thanks to their application, the quality of life of patients suffering from these situations (on a daily basis or occasionally) is improved. **URITAM** tampons are not medicinal tampons.

**Before using URITAM tampons, adjust their size by using the Starter Set A or B. Each set consists of 3 tampons of different sizes, which allows you to choose the right one. This is especially important because the wrong kind of tampon will not provide the expected protection and comfort.**

**To learn more about the URITAM starter kit, visit [www.uritam.com](http://www.uritam.com)**

## Method of use:

- Always wash your hands before applying the tampon.
- In order to maintain the highest level of hygiene, each tampon has been packaged individually. After removing the product from the box, remove the plastic packaging protecting the tampon.
- The unwrapped tampon has a hard foam consistency and cannot be applied in this condition.
- To prepare the tampon for use, immerse it in clean, warm water for about 1 minute, so that it becomes soft in its entire volume.
- Then, squeeze the tampon thoroughly to remove the water.
- If the tampon is inserted too far, it will press against the urinary bladder instead of the wall of the urethra, which will not prevent urine leak. To make sure it works, do not insert it too far, as shown in the video at [www.Uritam.com](http://www.Uritam.com) or in the picture in the leaflet.
- It is important to remember that the string for removing the tampon should not be wrapped around the tampon; the end of the string should be outside the vagina.
- A properly placed tampon should not cause irritation or discomfort. If these symptoms are observed, its position should be adjusted slightly. There may be a feeling of gentle pressure.
- The tampon does not need to be removed before urination.
- To remove the tampon, gently pull the string.
- In the event of damage or rupture or if the string is not unrolled, remove the tampon with your fingers, grabbing its end (remember that you must have clean hands when doing this).

- If you cannot remove the tampon yourself, please contact your gynaecologist.
- The removed tampon should be packed in a small bag and thrown into the rubbish.

## Important:

- A single tampon can be used for up to 12 hours.

## Use during pregnancy:

- The use of tampons during pregnancy should be consulted with a gynaecologist.

## Use by breastfeeding women:

- Tampons are not used only during the postpartum period.

## Contraindications:

- inflammation of the vagina and vulva,
- menstruation and other abnormal bleeding from the genital tract,
- allergic reaction to tampons.

## Side effects:

In the case of changes in mucus, inflammation, burning or similar symptoms, remove the tampon and consult a gynaecologist.

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